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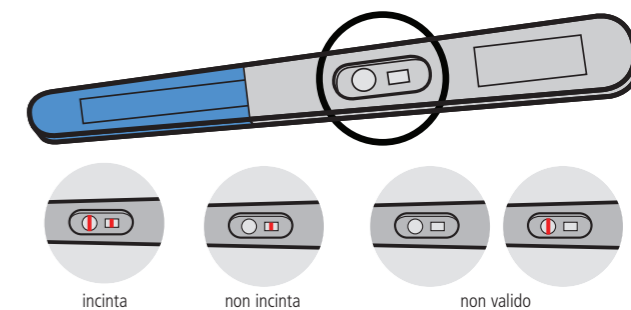
Axaclear Pregnancy test
Can be used already 10 days after a possible conception.

Pregnancy test to detect the hormone hCG in the urine. In-vitro diagnostic product for selftesting. Detection limit: 20 mIU of hCG/ml urine.

Pregnant or not? The **Axaclear** Pregnancy test answers this question already 10 days after a possible conception – and takes only 2-5 minutes.

Instructions for use

- Open the foil bag immediately before performing the test.
- Remove the protective cap.
- Grip the test strip in one hand and hold it in the urine stream for a few seconds. Alternatively, you can collect the urine in a beaker and dip the tip of the test strip into the urine for about 5 seconds.
- Then put the protective cap back on the test strip and lay it on a flat surface.
- Look closely and after a few seconds you will see the moisture running along in the test strip windows.
- Read the results 2-5 minutes after use. Caution! Please note that the result should not be read later than 10 minutes.

Evaluation

Pregnant: A line is visible both in the rectangular control window and in the round test window.
Not pregnant: A line is visible only in the rectangular control window.

Note

If no line is visible after about 5 minutes of using the test or if a line is only visible in the round window, the result is invalid. Read the «Safety» notes in these instructions. The two lines may differ in colour intensity and width. This is normal! If you are still unsure about the evaluation after 2-5 minutes, read the test again after another 5 minutes. It often happens that the line in the round window becomes more coloured during this time so that it is easier to evaluate the test.

Performance

Reliability

The **Axaclear** Pregnancy test has a scientifically proved precision of >99%.

Test procedure

Even in the early phase of pregnancy the body starts to produce the hormone hCG (human Chorionic Gonadotropin). This hormone is also excreted in the urine and can be detected with the **Axaclear** Pregnancy test.

Medicines

Taking medicines generally does not affect the test result. The only exceptions are medicines which contain hCG. They can falsify the result.

Safety

The **Axaclear** Pregnancy test is particularly reliable. Nevertheless, in rare cases the result may be unclear or difficult to interpret.

If, for example, the test line is particularly faint, please perform a new test after 3 days and use your morning urine for it. If you are pregnant, 2 clearly visible lines will then be seen. If the test is positive, please make sure to consult your doctor.

If the test result reads «not pregnant» but your period still does not come, we advise you to perform another test after 3 days. If this test is also negative and your period continues to be missed, please consult your doctor.

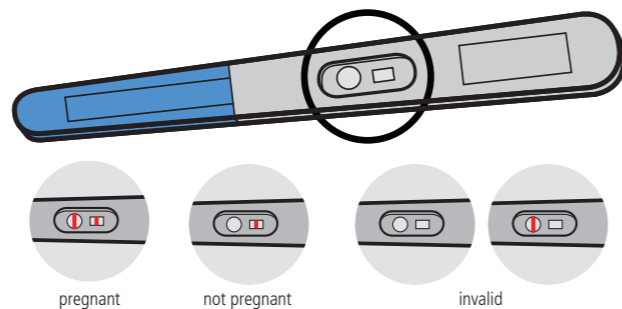
Similarly, if the test result is positive, please also contact your doctor. He/she will give you important information and advice for the optimal course of your pregnancy.

Note

- Keep out of the reach of children
- Do not take the test by mouth
- Do not use the test if the foil bag is damaged
- Do not use the test after its expiry date
- Dispose of the used test strip with the household rubbish
- Additional materials required: a clock and, if applicable, a clean, dry beaker for collecting the urine.
- Some of the materials used (e.g. antibodies) are potentially infectious materials but there is no risk from them if the test is performed as instructed.

Technical data

Product name: **Axaclear** Pregnancy test
Active substance: Monoclonal anti-hCG antibodies
Test material: Fresh human urine, no pretreatment, no storage
Purpose of use: In-vitro diagnostic test for personal use to detect hCG in human urine
Performance features: Detection limit: 20 mIU hCG/ml
Measurement range: 20 mIU hCG/ml – 600 IU hCG/ml
Diagnostic sensitivity >99%
Diagnostic specificity >99%
Storage: In the original pack at room temperature (+2° to +30°C)
After opening the foil bag, the test should be used immediately.
Test principle: The urine applied to the test strip is absorbed into the strip by capillary force. If hCG is present in the urine there is an immunological reaction between the hCG and the antibodies present in the test strip, leading to the formation of a red line (result line). Other antibodies form the control line, which demonstrates that the test strip has worked properly. The method is called immunochromatography.
Date: 09/03/2015, Rev. 3.0 Axapharm

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